



Hyfrecator® 2000 Power Settings Guide

Lesions	Power Setting (watts on low)	Type of Electrode
BENIGN		
Angiomas (cherry)	2-2.5	Sharp or blunt
Angiomas (spider)	2-2.5	Sharp or needle
Condyloma acuminata	12-18	Dull
Dermatosis papulosa nigra	2-2.5	Sharp or blunt
Pyogenic granulomas	16-20 or switch to high	Blunt
Sebaceous hyperplasia	2-2.5	Blunt
Seborrheic keratosis	10-14	Blunt
Skin tags (acrochordons)	2-2.5	Sharp
Syringomas	2-2.5	Sharp
Telangiectasias	2-2.5	Sharp or needle
Verrucae vulgaris	12-18	Blunt
Verrucae plana	12-18	Sharp or blunt
MALIGNANT		
Basal cell carcinoma	16-20	Blunt
Squamous cell carcinoma	16-20	Blunt

Disclaimer: Every patient and every electrocautery unit is different. These numbers are just suggestions and each clinician must find the best settings based on experience with their patients and unit.

This table was published in *Dermatologic and Cosmetic Procedures in Office Practice*, vol. 1, Ed. Dr. Richard Usatine, Table 14-1, Chapter 14, Copyright Elsevier Inc, 2012.

Customer Service: 1-800-448-6506
International Sales: 1-315-797-8375
Fax: 1-800-438-3051

CONMED.com
info@mail.conmed.com

525 French Road, Utica, NY 13502

©2016 CONMED Corporation, MCM2014007 Rev A 8/16