

OPTIONAL HEAT AND MASSAGE


Our FR566, FR577 and FR587 Clinical Care Recliners are now available with Optional Heat and Massage. You want your patients to be as comfortable as possible before, during, and after treatment. One of the most common complaints from patients is that they feel cold during treatment. Lumex Clinical Care Recliners are now available with an optional Heat and Massage feature to help your patients stay warm and comfortable! Employing an easy-to-use single switch to independently control both the Heat and Massage feature, the switch is conveniently mounted on the right side of the recliner for easy access / use by both patient and caregiver.

Two heating elements, one in the seat and one in the back, provide gentle heat for the patient. Three Heat Settings allow patients to control just how much warmth they need. The heat feature has an automatic shut-off to help ensure that the patient is always in control of their comfort. Heat automatically shuts off after one hour of continuous use at the same level.

Many patients appreciate a gentle massage during treatment. The massage function offers a four massage settings that control the intensity and location of the massage to ensure that the patient receives the most appropriate level of massage. Massage automatically shuts off after fifteen minutes of continuous use at the same level. The Heat and Massage Option is powered by a low-voltage transformer with an eight-foot hospital-grade 120V cord. The Lumex Heat and Massage Unit meets UL60601-1 Standards for electric medical devices.



Massage motor

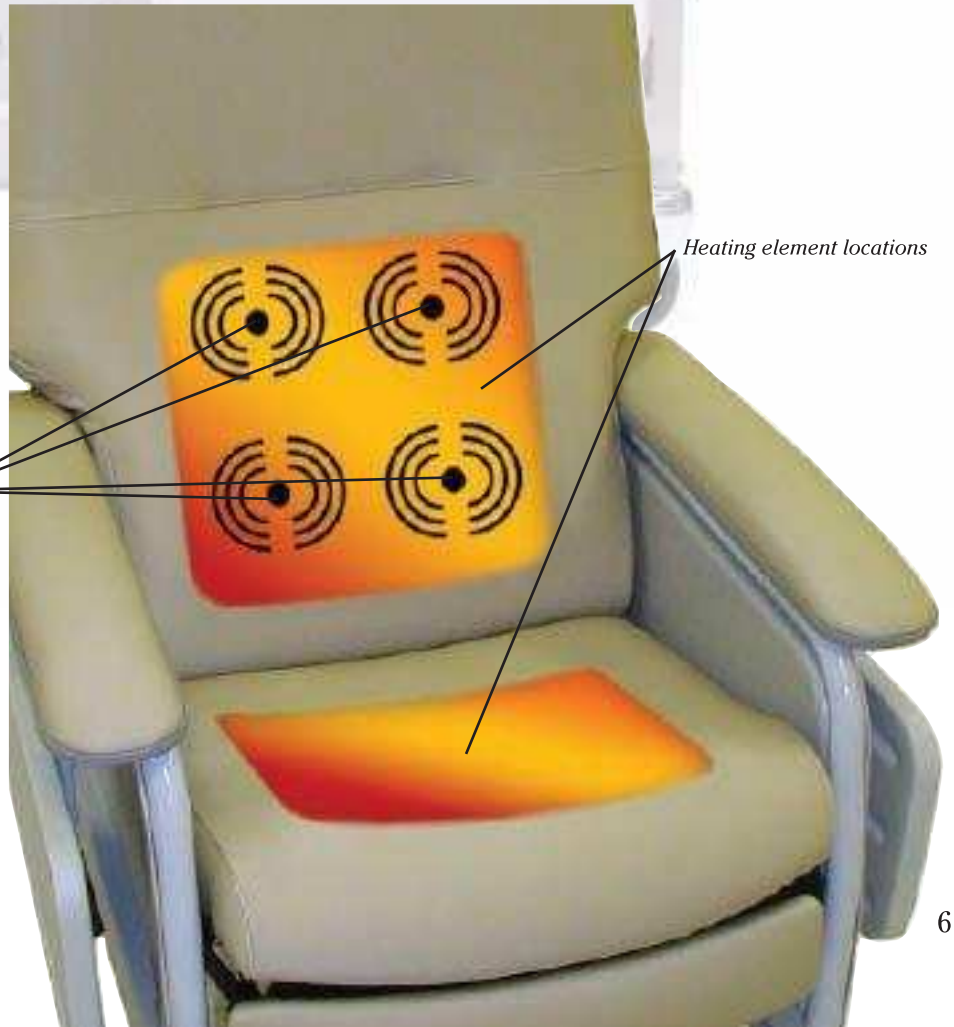

Massage motor locations

Heating element locations