

CLINICAL CARE SEATING



HIGH PERFORMANCE UPHOLSTERY

eco^{sense} BY SPRADLING GREEN COLLECTION

GUARDIAN eco^{sense} BY SPRADLING

SAVOY eco^{sense} BY SPRADLING

CASSARA eco^{sense} BY SPRADLING

PERFORMANCE COLLECTION

DOLCE⁺



Composition & Backing

	DOLCE	PATRIOT PLUS	RENAISSANCE	GUARDIAN	SAVOY	CASSARA
PVC	—	■	■	■	■	—
PU	■	—	—	—	—	—
TPE	—	—	—	—	—	■
Polyester Backing (100%)	—	—	—	—	—	■
Polyester Backing (100%) Hi-Loft ² ™	—	—	■	—	■	—
Polyester Backing (100%) Ultra Hi-Loft ² ™	—	—	—	■	—	—
Polyester / Cotton Backing	—	■	—	—	—	—
Rayon Backing (100%)	■	—	—	—	—	—
Weight (oz/liner yd)	15	34	32	30	30	22
Roll Size (yd)	30	30	30	30	30	30
Roll Width (inch)	54	54	54	54	54	54

Flammability Codes *

	DOL	PAT	REN	GUA	SAV	CAS
NFPA 260 - Cover Fabric - Class 1	—	■	■	■	■	■
UFAC Fabric - Class 1	—	■	■	■	■	■
CAL TB 117-2013	■	■	■	■	■	■
FMVSS 302	—	■	—	—	—	■
BS 5852 - UK - Source 0	■	—	—	—	—	—

Finishes & Performance

	DOL	PAT	REN	GUA	SAV	CAS
PBG⁺ PERMABLOK[®] PERMAGUARD[®]	—	—	—	■	■	—
PERMABLOK[®] ADVANCED VINYL PROTECTION	—	■	■	—	—	—
SUREGUARD[™]	—	—	—	—	—	■
Abrasion Resistance (k) CFFA-1 ASTM D-4157-02	100	100	100	100	100	100
Antibacterial Protection (hours)	—	—	—	1k	—	—
Blue Wool (hours) AATCC TM 16-3 / NTC 1479 / CFFA-2a	—	1k	1k	1k	1k	1k
QUV Stabilized Pigments (hours)	650	—	—	—	—	—
Wyzenbeek Cotton Duck	#10	#10	#10	#10	#10	#10
Cold Crack (F)	-10°	-20°	-20°	-10°	-10°	-10°

Applications

	DOL	PAT	REN	GUA	SAV	CAS
Recliners	■	■	■	■	■	■
Physician & Laboratory Stools	—	—	—	—	—	—
Exam Tables & Replacement Tops	—	—	—	—	—	—
Blood Donor & Phlebotomy Chairs	—	—	—	—	—	—
Wall Saver Guest/Patient Seating	—	—	—	—	—	—

* This term and any corresponding data refer to the typical performance in the specific tests indicated and should not be construed to imply the behavior of this or any other material under actual fire conditions

CLINICAL CARE SEATING

published October 2024



Color Variations Swatches are digital reproductions. Perceived texture and coloring may vary on finished product. Monitor and printer settings will effect appearance of color.

Fabric Availability current at time of publication and subject to change without notice. Contact Customer Service for most current availability +1 770.368.4700 | cs@grahamfield.com

PREMIERE COLLECTION


PATRIOT PLUS

NEO



RENAISSANCE

NEO

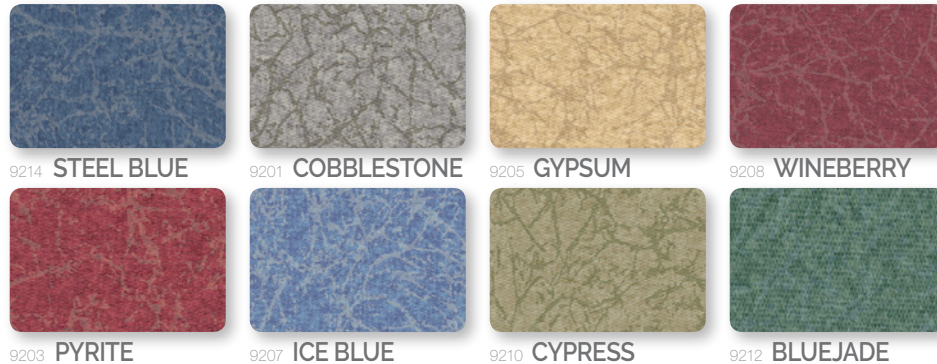

Disinfectants

If you need to disinfect the product because it has come into contact with organic material (blood, urine or fecal matter), check the disinfection guide for appropriate cleaning products. Clean with a clean white cloth or fabric. If you choose to use a bleach solution, only use 0.5% concentration (1 part bleach to 10 parts water). Rinse and wipe with a clean damp cloth to get rid of any excess and dry.

For Color Transfer

Our products don't transfer onto other materials, but some clothing items (like jeans, microfibers and dark fabrics) could transfer their color to the coated fabrics, depending on how they are dry-cleaned. This happens more with dampness and higher temperatures and is irreversible. Neither GF Health Products, Inc. nor Spradling International, Inc.[®] is responsible for any damages caused in this manner. Please check compatibility when using this product in combination with painted or varnished surfaces.



SPRADLING[®]

Cleaning Guides specific for each finish technology. | www.spradling.group

Guide to Healthcare Surface Disinfection for the Real World. | www.spradling.group

GF Health Products, Inc. is an ISO 13485:2016 / MDSAP Certified Company

Color variations may occur due to monitor or printer settings.

+1 770.368.4700

Information contained herein is subject to change without notice. The most current and complete product information can be found on our website.

www.grahamfield.com



© 2015, GF Health Products, Inc. All Rights Reserved. ©2021 Spradling International, Inc.[®] Spradling, Neo, EcoSense, PBG+, Permguard, PermaBlock, and Sureguard are trademarks of Spradling Holdings, Inc. Lumex is a registered trademark of GF Health Products, Inc.

CLINICAL CARE SEATING

Care and Cleaning Guide[†]



We recommend cleaning at least once a week to extend the lifespan of the upholstery, which will prevent dirt and contamination buildup and permanent staining. In the event of a spill, wipe off any excess right away with a dry white cloth or fabric, but careful not to spread to remaining surface.



Clean the whole surface in a circular motion using pH neutral liquid hand soap and water (1 part soap to 9 parts water). Check the Approved Cleaners guide for recommended cleaning products. Wipe with a clean damp cloth to get rid of any excess soap. Repeat as needed and then let dry.



[†] If there is any buildup that cannot be removed with soapy water, a mixture of 1 part isopropyl alcohol to 9 parts water can be used as a last resort. (Remember, using alcohol on a regular basis can affect the properties of the product.)



Once a month, you can use a soft-bristle brush for a deeper cleaning. Remember, light colors need more care, more often.

[†] This section of cleaning guide does not apply to Dolce.

Color Variations

Swatches are digital reproductions. Perceived texture and coloring may vary on finished product. Monitor and printer settings will effect appearance of color.

Fabric Availability

Current at time of publication and subject to change without notice. Contact Customer Service for the most current availability. +1 770.368.4700 cs@grahamfield.com

Note

This guide has recommendations for properly using and cleaning these products. Users are responsible for using and cleaning the coated fabrics safely with products that don't affect the surface, composition or, as a result, their performance and lifespan. The information in this cleaning guide refers to performance in specific tests conducted under laboratory conditions. Cleaning results may vary depending on the size of the stain, cleaning product used and time the surface is exposed. This information is not a guarantee and does not relieve the user from the responsibility of the proper and safe use of the product and referenced cleaning agents. Neither Spradling International, Inc.[®] nor GF Health Products, Inc. have a relationship with the manufacturers of any cleaning products.

published October 2024

graham field

GF_1400068Rev124