

Indications	Seconds of Application
Acne	4-6
Keratosis Actinica	5-7
Age Pigments	1-2
Basilioma	8-9
Cavernous Angioma	4-6
Granuloma Annulare	5-6
Hemangiolympangioma	6-8
Juvenile Warts	3-4
Keloids	4-6
Senile Keratoma	7-8
Condyloma	8-12
Lentigo Maligna	4-5
Lentigo Senile	2-3
Nevus Pigmentosus	8
Plantar Warts	15-20
Solar Keratosis	4-5
Seborrhoeic Warts	6-8
Verruca Vulgaris	8-10
Soft Fibroma	4-5