



Attaching / replacing the cartridge into the control mechanism is aided by a pin\* which is easily inserted into the control mechanism

1. Insert the metal pin into the hole on the control mechanism.



2. Hold the control mechanism at the pin position and not at the on/off trigger. This helps grip the device without damaging the trigger.



3. Screw the cartridge into the control mechanism in a clockwise direction both quickly and completely to ensure proper functionality.

4. Care should be taken to align the cartridge and control mechanism so that gas does not inadvertently escape.



5. Small amounts of gas escaping from the cartridge while being attached indicates poor alignment of the cartridge to the control mechanism.

6. Remove the pin.